



A multi-center case study to evaluate HLA typing resolution using novel SBTessenz® assays suitable for donor registries

Introduction

Many countries have set-up donor registry HLA-typing programs to facilitate unrelated hematopoietic stem cell transplantation. The NMDP requires that each newly recruited donor is typed at the intermediate resolution level for HLA-A, -B, -C and -DRB1. For bone marrow transplantation, HLA-DQB1 is also required. We developed novel SBTessenz® assays that enable unique identification of HLA-A, -B, -C, -DQB1 and -DRB1 gene regions that code for the antigen recognition site using sequencing-based typing (SBT) (Figure 1). The SBTessenz® assay strategy accommodates intermediate resolution in the first round of sequencing through application of a specified set of group-specific sequencing primers (GSSPs) in addition to the Core sequencing primer set (Figure 1).

Aim

In this multi-center study the performance of the SBTessenz® assays was evaluated by BFR Gene Diagnostics, UMASS Memorial Health Care and Leiden University Medical Centre, using different gDNA isolates, including blood, buccal swabs, and cell-line material of diverse DNA qualities.

Results

Strong HLA locus-specific amplicons of the expected sizes were obtained for the different sample panels tested: LUMC (n=192), BFR (n=96), and UMASS (buccal swabs: n=42) using the SBTessenz® assays (Figure 2). Sequences with high quality scores were generated for all amplicons (Figure 3). Typing results obtained (total of n=330 samples) using the HLA-A, -B, -C, -DQB1, and -DRB1 SBTessenz® assays were concordant with the pre-typing results provided by the centers.

The resolution levels obtained for the Chinese and LUMC sample panel were analyzed for the SBTessenz® assay strategy (Core plus GSSPs) versus a generic approach (Core only) (Figure 4). The percentage of genotype ambiguities drops significantly for all loci when applying the SBTessenz® strategy (Figure 4).

When the Common and Well-documented (CWD) filter is applied, thereby ignoring CWD +/- ambiguous combinations, the level of genotype ambiguities reaches: HLA-A [19%], HLA-B [50%], HLA-C [26%], HLA-DRB1 [17%], and HLA-DQB1 [5%] (Caucasian); HLA-A [19%], HLA-B [13%], HLA-C [7%], HLA-DRB1 [13%], and HLA-DQB1 [4%] (Chinese). Further analysis, ignoring CWD +/- and CWD +/- ambiguous combinations, show that the majority of samples have a single common genotype (Figure 5).

Figure 1. Locations of EAP participants

Exons 2 and 3 are amplified for HLA-A, -B, and -C, exon 2 for HLA-DQB1 and -DRB1 (red arrows). Sequencing reactions are performed using the core sequencing primers (blue arrows) and the SBTessenz® specified GSSPs (green arrows).

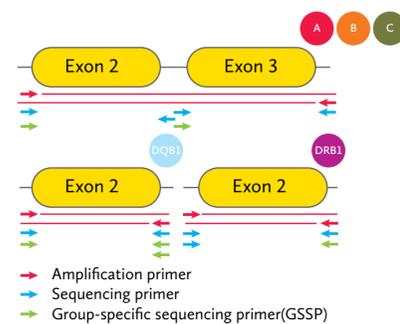


Figure 2. Robust HLA locus-specific amplification on multi-center sample panels

Agarose gel-picture depicting a representation of amplicons obtained from sample panel LUMC (n=192), BFR (n=96), and UMASS (n=42) using SBTessenz® HLA-A, -B, -C, DQB1, and -DRB1 typing assays.

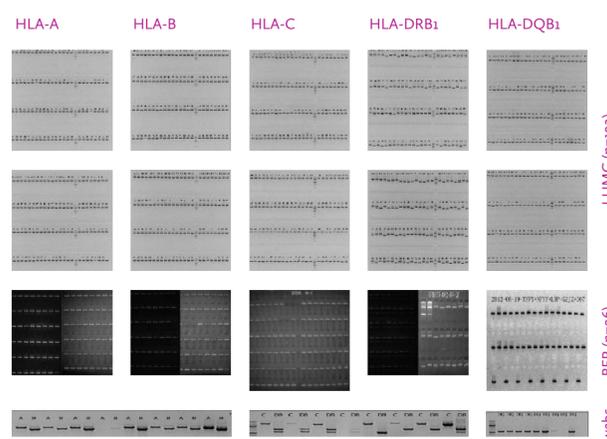


Figure 3. HLA-DRB1 sequence data showing high quality sequencing scores on buccal swab isolates.

Example of HLA-DRB1 sequence data obtained from exon 2 forward and reverse sequencing primers, as analyzed by the SBTengine® software.


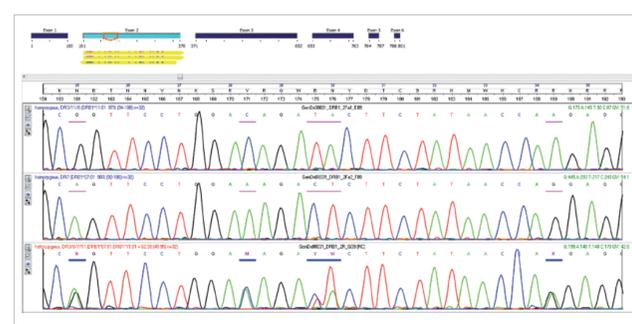


Figure 4. Enhanced genotype resolution when applying the SBTessenz® strategy

Genotype resolution was evaluated at the second field level for the Chinese and Caucasian sample panels. Comparison was made between the SBTessenz strategy (Core plus GSSPs) versus a generic approach (Core only). "Generic CWD / SBTessenz CWD" represents the resolution levels when CWD +/- ambiguous allele combinations are ignored. (CWD: source Cano et al. 2007).

Caucasian case study

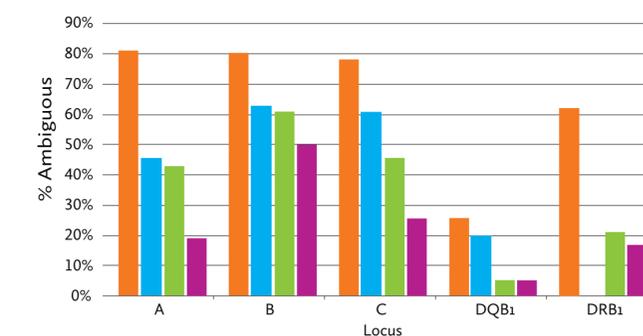

Chinese case study

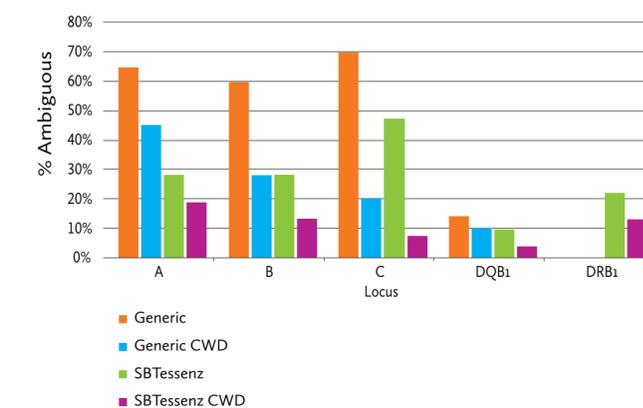


Figure 5. Genotype resolution when applying SBTessenz

Genotype resolution is shown for the number of samples that have a common genotype when CWD +/- and CWD +/- combinations are ignored.

Sample panel	Locus	Single common genotype CWD +/-
Caucasian	HLA-A	98%
	HLA-B	86%
	HLA-C	99%
	HLA-DQB1	97%
	HLA-DRB1	97%
Chinese	HLA-A	99%
	HLA-B	99%
	HLA-C	99%
	HLA-DQB1	98%
	HLA-DRB1	97%

Key Features SBTessenz®

- Unique identification of the antigen recognition domains for HLA-A, -B, -C, -DQB1 and -DRB1
- One-step DNA sequencing strategy with enhanced ambiguity resolution
- Robust performance on different gDNA materials including buccal swabs
- Plate set-up for high throughput of large sample panels
- Reduced technical effort for improved accuracy
- Low-cost typings suitable for donor registries

Conclusions

The multi-center study shows that the SBTessenz® assays identifies the antigen recognition site of HLA-A, -B, -C, -DQB1, and -DRB1 in an accurate and robust manner and provides enhanced genotype resolution in one DNA-sequencing step. The SBTessenz® assays are good complementary assays to the full gene SBT approach as it enables reliable HLA typings on buccal swab isolates and gDNA of lower quality.

Jeroen Adema, B.ASc¹, Nienke Westerink, PhD¹, Jessica Gatulis, PhD², Neng Yu, PhD², Wang Dan, B.ASc³, Xiangjun Liu, PhD³, Dave Roelen, PhD⁴, Frans Claas, PhD⁴, Wietse Mulder, PhD¹ and Erik Rozemuller, PhD¹.

1. GenDx, Utrecht, The Netherlands
2. Histocompatibility, UMASS Memorial Health care, Worcester, United States
3. BFR biotech, BFR Gene Diagnostics, Beijing, China
4. Immunohematology and Blood Transfusion, Leiden University Medical Centre, Leiden, Netherlands

personalising diagnostics